

## Full-Cast



**Full-cast** crowns have long been a popular choice due to their clinical success, biocompatibility, and ease of use. Durable yet gentle on opposing dentition, full cast provides ideal fit, form, and function.

CAD/CAM fabrication ensures a high-quality, consistent fit and finish. No other restoration matches the reliability and durability of full cast

### Indications

Full-cast gold crowns are indicated for crowns, veneers, inlays, onlays, and bridges.

### Contraindications

Full-cast gold crowns are contraindicated for partials and implants.

### Prep Guides

Inlays and onlays can also be fabricated as a full-cast restoration. Feather-edge margin preparations are indicated for full-cast restorations, but any margin preparation may be used.

### Cementation

- Panavia 21 (Must be tin plated if precious metal is used)
- Glass ionomer cement (GC Fuji, GC America)
- Zinc Phosphate Polycarboxylate Resin Ionomer cement (RelyX, 3M ESPE)