

How to Prescribe a New Denture



Step One: Impression

Use current impression technique.



Step Two: Model

R-Dent will pour the model and create bite rims based on the provided impression.



Step Three: Scan

R-Dent will scan traditional bite rims and the model/impression.



Step Four: Digital Design

Creation of digital set-up and design.



Step Five: Try-In

Receive the final printed denture base with anterior denture teeth set and bite blocks in the posterior.

Please note: This step can be skipped if you chose to go straight to final denture without try-in.



Step Six: Final

Upon completion of try-in or approval from the digital design in step four, R-Dent will fabricate and deliver the final denture.



INsure[™]
DIGITAL DENTURE

Only \$199 Per Arch

- 6-Year Digital Record
- Available in Silver Series
- Fast Turn Time with High-Quality
- Utilizes Traditional Bite Rims
- Final Can be Delivered at 3rd Appointment

Powered By: **Carbon**

Prescribe INsure™ Silver Series Digital Dentures Today!
Visit rdentlab.com/prescribe-insure to get started!

7490 Bartlett Corporate Cove West
Bartlett, TN 38133 | 901-372-8020
customerservice@rdentlab.com
www.rdentlab.com

Prescribing INsure™ Digital Dentures for Every Edentulous Case



Intraoral Scan of Denture with Wash Reline



Printed Try-In with Bite Blocks



Printed Final Denture

PDF approval is available before final printing upon request.

Replacing a Current Denture

Step One: Impression

Use preferred impression technique, whether that be through traditional materials or an intraoral scan, on denture with wash inside.

What to Send:

Traditional: Denture with wash, opposing model, and bite.

Digital: Scan of existing denture with wash reline, bite scan, opposing scan, and facial photos of high and low lip line.

Step Two: Final

R-Dent will provide the final INsure™ Digital Denture.

Immediate Denture: Traditional

Step One: Impression

Provide a traditional impression.

Step Two: Scan

R-Dent will scan traditional bite rims and the model/impression.

Step Three: Digital Design

Creation of digital set-up and design.

Step Four: Final

Receive final denture from R-Dent.

Immediate Denture: IOS File

Step One: Impression

Provide scan of upper and lower arches and bite scan. For submission, view our DI protocols by visiting rdentlab.com/go-digital.

Step Two: Digital Design

Creation of digital set-up and design.

Step Three: Final

Receive final denture from R-Dent.